

AUTO-FREEWHEEL REMOVAL

- 1) Remove front wheel from tryke if already installed.
- 2) Located on each side of the wheel there are plastic hub-caps.
- 3) Behind the hub-cap on the right side of the wheel is the workings of the auto-freewheel.
- 4) Pull the hub-cap back far enough to see the spring and the ratchet assembly.
- 5) Using the screw provided, insert the screw between the ratchet assembly (Unit with spring around it) and the inside diameter of the wheel.
- 6) Using a screw driver, screw in the screw until it is fully inserted.
- 7) Install the front wheel back on the tryke and you are finished.